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## myMIRACLE™ | INSTRUCTION FOR USE Fecal Incontinence Insert

**Description:** The Fecal Incontinence Insert is a soft, flexible, liquid-filled insert designed to prevent fecal incontinence. The device is non-sterile and for single use only.

**Indications for use:** The Minnesota Medical Technologies Fecal Incontinence Insert is indicated for the management of Accidental Bowel Leakage (ABL) due to bowel incontinence. The rectal insert is designed for self-insertion to seal and help prevent the involuntary leakage of stool from the rectum.

### Risks:

- Possible mild irritation or discomfort of the anus or anal canal
- Possible migration or displacement of the insert into the anal canal and rectum
- Unexpected expulsion

**Benefits:** The Fecal Incontinence Insert is a rectoanal insert which is intended to assist adult patients and caregivers in the management of fecal incontinence. The device is designed to prevent involuntary leakage of fecal matter through the anal canal.

**Sizing:** The Fecal Incontinence Insert comes in multiple sizes. It is recommended that you begin with the smallest size unless otherwise instructed by your physician. Upon engaging in daily activities, if you experience leakage or the device comes out unexpectedly, try a larger size.

**Length of Wear:** The Fecal Incontinence Insert may be worn continuously. A single insert may be worn until your next bowel movement or changed as you see fit. After one insert has been expelled, you may replace it with another. Only one insert may be used at a time.

### Insertion:

- Wash your hands.
- Open the Fecal Incontinence Insert package and remove the device.
- Place yourself in a relaxed, comfortable position, such as lying on a bed. Grasp the applicator and device retainer. (A) Align the device with the anus. If preferred, additional lubrication may be added to the device and/or anus.
- Relax your muscles and gently insert the Fecal Incontinence Insert until the retainer rests on the anus (B) (C)
- Withdraw the applicator and discard the applicator and packaging with the normal trash (D)
- Wash your hands.

**Removal:** The Fecal Incontinence Insert is designed so that it may be expelled with a normal bowel movement or manually removed. For convenience, it is recommended that the device be removed before a bowel movement. To manually remove:

- Grasp the external retainer and gently pull to remove the device. (E)
- Discard the device with the normal trash. Flushing of the device or device applicator is not recommended.
- Wash your hands.

**Warnings, Precautions, and Contraindications:** Fecal Incontinence may be medically treated. The Fecal Incontinence Insert is not intended to replace surgical options to treat accidental bowel leakage. Consult your physician for professional advice.

- The Minnesota Medical Technologies Fecal Incontinence Insert is contraindicated for use in the pediatric population.
- This device is non-sterile and for single use only. Reuse of the device could result in device breakage.
- Do not use if the package is open or damaged.
- Do not use if you have a silicone allergy.
- Do not use with suppositories.

Discontinue use of the Fecal Incontinence Insert and consult your physician if you have:

- Bloody stool or bleeding from the rectum
- Bleeding hemorrhoids
- Anal/rectal pain, irritation, spasms, tearing, or fissure
- Displacement of the Fecal Incontinence Insert into the anal canal or rectum which has not been expelled with the next bowel movement
- Undergone rectal surgery within the last 4 weeks.

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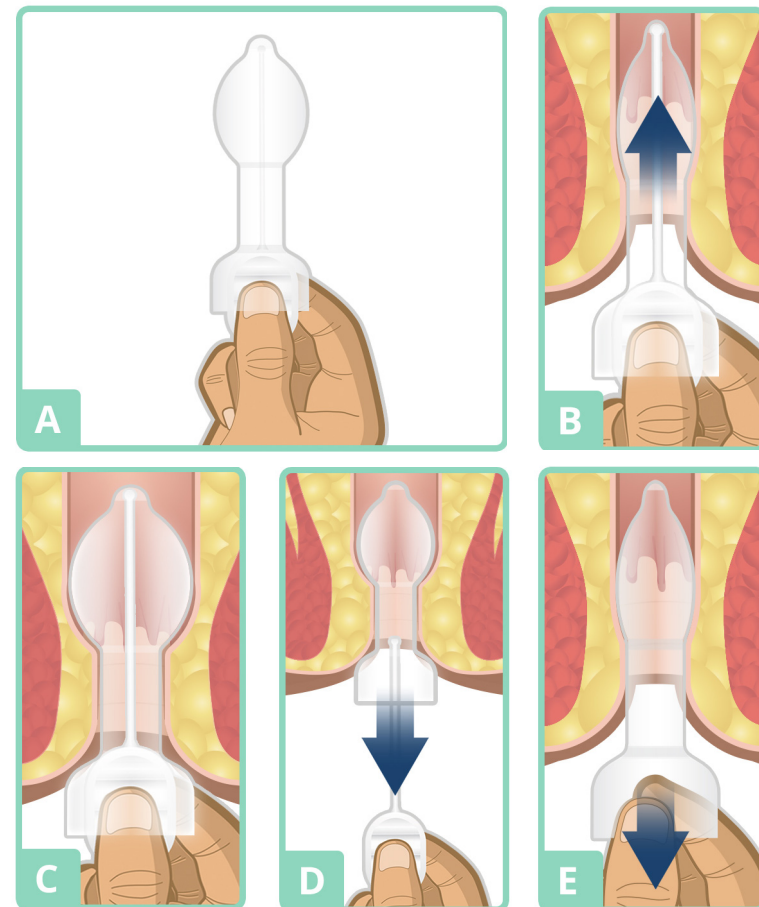
**Rx Only** Caution: Federal law restricts this device to sale by or on the order of a physician

Do not reuse

Consult instructions for use

Do not use if package open or damaged

Use by date

**REF** Product reference number/Catalog number

Manufacturer

**EC REP** Authorized representative in the European Community

**CE 0123** European conformity

**LOT** LOT number

Patent Pending  
myMiracle is a trademark of Minnesota Medical Technologies  
© 2018 Minnesota Medical Technologies  
540002(Rev 1), 111109, 2018-04

Minnesota Medical Technologies  
2446 Henry Road Northwest  
Stewartville, MN 55976  
507-533-0366  
www.mnmedtech.us

**EC REP**

MPS Medical Product Service GmbH  
Borngasse 20  
Braunfels 35619  
Germany

**CE 0123**