DK

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ADVARSLER, FORHOLDSREGLER OG

SYMBOLS GLOSSARY:

	Caution: Federal law restricts this device to sale by or on the order of a physician	
	Do not reuse	
	Consult instructions for use	
	Do not use if package open or damaged	
	Use by date	
	Product reference number/Catalog number	
	Manufacturer	
	Authorized representative in the European Community	
	European conformity	
	LOT number	
	Medical device Medizinprodukt Dispositif médical Medisch hulpmiddel Dispositivo medico	Product sanitario Medicinteknisk produkt Medicinsk udstyr Medisinsk utstyr Lääkinnällinen laite
	Importer Importeur Importateur Importatore	Importador Importör Importeur Importero Maahantuoja

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LÄNGE:

The Fecal Incontinence Insert comes in multiple sizes. It is recommended that you begin with the smallest size unless otherwise instructed by your physician. Upon engaging in daily activities, if you experience leakage or the device comes out unexpectedly, try a larger size.

TIKKEDEL:

The Fecal Incontinence Insert may be worn continuously for up to 24 hours. A single insert may be worn until your next bowel movement or changed as you see fit. After one insert has been expelled, you may replace it with another. Only one insert may be used at a time.

INDSÆTTNING:

- Wash your hands.
- Open the Fecal Incontinence Insert package and remove the device.
- Place yourself in a relaxed, comfortable position, such as lying on a bed. Grasp the applicator and device retainer. (A) Align the device with the anus. If preferred, additional lubrication may be added to the device and/or anus.
- Relax your muscles and gently insert the Fecal Incontinence Insert until the retainer rests on the anus (B) (C).
- Withdraw the applicator and discard the applicator and packaging with the normal trash (D).
- Wash your hands.

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Withdraw the applicator and discard the applicator and packaging with the normal trash (D)

Wash your hands.

RETIRADA:

The Fecal Incontinence Insert is designed so that it may be expelled with a normal bowel movement or manually removed. For convenience, it is recommended that the device be removed before a bowel movement. To manually remove:

- Grasp the external retainer and gently pull to remove the device. (E)
- Discard the device with the normal trash. Flushing of the device or device applicator is not recommended.
- Wash your hands.

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Fecal Incontinence may be medically treated. The Fecal Incontinence Insert is not intended to replace surgical options to treat accidental bowel leakage. Consult your physician for professional advice.

- The Fecal Incontinence Insert is contraindicated for use in the pediatric population.
- This device is non-sterile and for single use only. Reuse of the device could result in device breakage.
- Do not use if the package is open or damaged.
- Do not use if you have a silicone allergy.
- Do not use with suppositories.

DISCONTINUE USE OF THE FECAL INCONTINENCE INSERT AND CONSULT YOUR PHYSICIAN IF YOU HAVE:

- Bloody stool or bleeding from the rectum
- Bleeding hemorrhoids
- Anal/rectal pain, irritation, spasms, tearing, or fissure
- Displacement of the Fecal Incontinence Insert into the anal canal or rectum which has not been expelled with the next bowel movement
- Undergone rectal surgery within the last 4 weeks.

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